

## The BabySense™ feature for the Hugs® system

The Hugs system has always offered comprehensive security, with door protection, tamper protection, and continual supervision of each tag. Now we've added a new feature to help you ensure that your infants are fully protected.

The BabySense feature for the Hugs system notifies you if the Hugs tag is not securely on the infant, either because the tag was attached too loosely or due to weight loss. By reminding you in a timely manner to check the tag, the BabySense feature helps ensure that the Hugs tag is always securely attached and protecting the infant.

### How it works



The back of the Hugs tag with BabySense is black. This layer, together with the standard Hugs strap, detects the baby's skin. If the tag is too loose, it sends a message to the Hugs system software.

### Using the BabySense feature

#### Attaching the tag

The Hugs tag with BabySense is attached in exactly the same way as the current Hugs tag, using the same strap. Just check that the tag is snug and won't rotate around the baby's ankle.

#### Event and alarm information

If the tag is not securely attached to the infant, an event notice appears in the Hugs system software. The event clears automatically when the tag is tightened, and only escalates into an alarm if no action is taken.

### Features

- **Helps you protect your infants**

BabySense tells you if the Hugs tag is not securely attached to the infant.

- **No false alarms**

The sensitivity of the BabySense feature is completely user configurable. You are only ever notified when you really need to know—when the tag is so loose that it could fall off.

- **Automatic and easy to use**

The BabySense feature is completely automatic, and builds on the Hugs tag's other automatic features like auto tag enrollment and automatic low battery notification.

- **Available to all users**

The BabySense feature is standard on all new Hugs systems, and is available as an upgrade to existing customers—all you need is a software upgrade and Hugs tags with BabySense.

**Call your authorized dealer, or Xmark at 1.866.55.XMARK to learn more.**



309 Legget Drive Ottawa, ON K2K 3A3 Canada

Telephone: 1.866.55.XMARK International: +1 (613) 592.6997 Facsimile: (613) 592.4296

Web site: [www.xmark.com](http://www.xmark.com) E-mail: [sales@xmark.com](mailto:sales@xmark.com)

© 2005-2008 Xmark Corporation. All Rights Reserved. Xmark and Hugs are registered trademarks of Xmark Corporation in North America. BabySense is a trademark of Xmark Corporation. All other company and product names may be trademarks of their respective companies. Printed in Canada. March 2008. MK50060 Rev 04.